


Leishmania Dipstick Rapydtest®

Qualitative detection of antibody
to visceral leishmaniasis in serum


APACOR

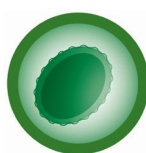

PARASITOLOGY
SINGLE USE IN VITRO
DIAGNOSTIC DEVICE


Leishmania Dipstick

Rapypdtest®

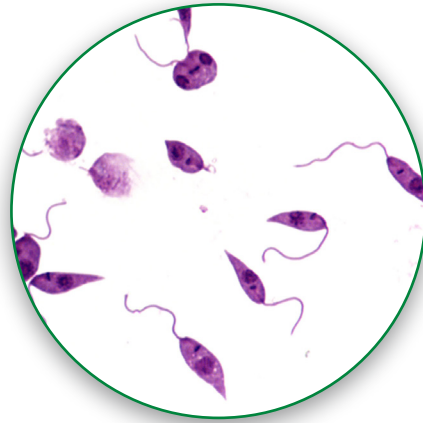

Performance benefits

- Ten minute one step test
- Rapid chromatographic immunoassay
- Detection in serum
- High sensitivity & specificity

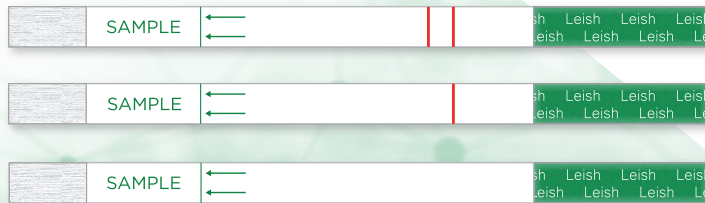

Positive result

Negative result

Invalid result

Field Studies

The Kalazar Detect™ Test for visceral leishmaniasis

SITE 1: Brazilian Study

Kalazar Detect Test compared to Microscopy

		+	-	
Kalazar detect™	+	115	0	
	-	13	59	
		128	59	187
Sensitivity	89.844		Specificity	100
STD Error	2.67			0
95% CL	(82.936,94263)			

SITE 2: Indian Study

Kalazar Detect Test compared to Microscopy

		+	-	
Kalazar detect™	+	225	14	
	-	0	190	
		225	204	429
Sensitivity	100		Specificity	93.137
STD Error	0			1.77
95% CL	(97.908,100)		(88.517,96.054)	

Code	Description	Pack size
1601	Leishmania Dipstick Rapypdtest®	40

A rapid, one step test for the qualitative detection of antibody to visceral leishmaniasis in serum

Visit www.apacor.com for all our latest information or contact us at sales@apacor.com