


H. pylori AG Rapydtest[®]

for the detection of
H. pylori AG in faeces

APACOR

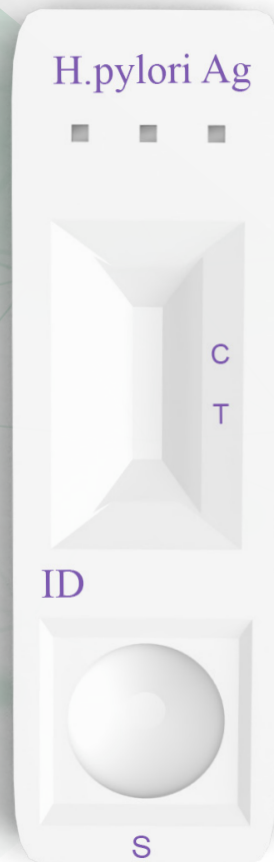

BACTERIOLOGY
SINGLE USE IN VITRO
DIAGNOSTIC DEVICE

H. pylori AG

Rapydtest®

Intended use

The H. pylori AG Rapydtest® is a lateral flow chromatographic immunoassay for the qualitative detection of H. pylori antigen in human faecal specimen. It is intended to be used by professionals as a screening test and as an aid in the diagnosis of infection with H. pylori.






Performance characteristics

324 faecal samples collected from subjects with symptomatic gastrointestinal disorders and non-gastrointestinal symptoms were tested with the H. pylori AG Rapydtest® and with the UBT as reference test.

Relative sensitivity - 94.4%

Relative specificity - 100%

Overall agreement - 97.8%

	Negative result
	Positive result
	
	Invalid result
	





**For liquid samples




2 drops
70-90ul



2 drops

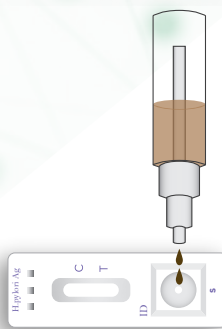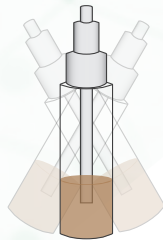

2 drops

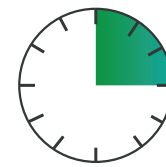

15 minutes

Code	Description	Pack size
1632	H. pylori AG Rapydtest®	25

Visit www.apacor.com for all our latest information or contact us at sales@apacor.com