
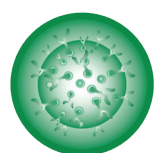


# Rotavirus/Adenovirus AG Rapydtest®

For the detection of  
Rotavirus/Adenovirus AG in faeces

APACOR

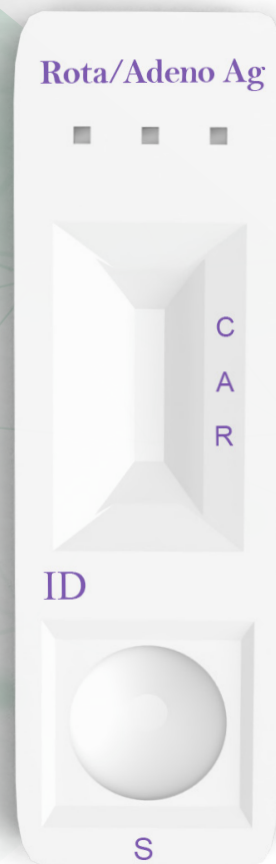

VIROLOGY  
SINGLE USE IN VITRO  
DIAGNOSTIC DEVICE

# Rotavirus/Adenovirus AG

Rapydtest®

## Intended use

The Rotavirus/Adenovirus AG Rapydtest® is a lateral flow immunoassay for the qualitative detection and differentiation of rotavirus and adenovirus antigen in human faecal specimens. It is intended to be used by professionals as a screening test and as an aid in the diagnosis of infection with Rotavirus and Adenovirus.

## Performance characteristics

### Rotavirus

**Relative sensitivity - 100%**  
**Relative specificity - 97.2%**  
**Overall agreement - 98.1%**

### Adenovirus

**Relative sensitivity - 100%**  
**Relative specificity - 97.9%**  
**Overall agreement - 98.1%**



	Negative result
	Positive result Adenovirus
	Positive result Rotavirus
	Positive result both
	Invalid result
	**Result is still valid if faint line is visible



\*\*For liquid samples



2 drops  
85-95ul



2 drops

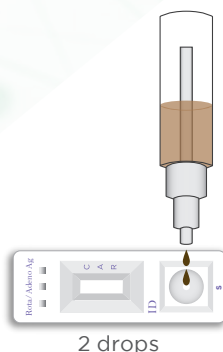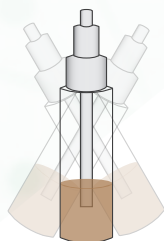

2 drops



15 minutes

Code	Description	Pack size
1640	Rotavirus/Adenovirus AG Rapydtest®	25

Visit [www.apacor.com](http://www.apacor.com) for all our latest information or contact us at [sales@apacor.com](mailto:sales@apacor.com)