
ParaBeads[®] IFU

To validate Mini/Midi Parasep[®] & Mini/Midi Parasep[®] SF

APACOR

Instructions for Use

REF 1850

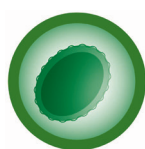PARASITOLOGY
SINGLE USE IN VITRO
DIAGNOSTIC DEVICE

ParaBeads[®]

INTENDED USE

To validate Mini/Midi Parasep[®] & Mini/Midi Parasep[®] SF.

N.B. This is not to be run with a stool sample.

CONTENTS PROVIDED IN PACK

1 vial containing 1ml red ParaBeads[®] solution should be sufficient for testing:

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SAFETY WARNINGS AND PRECAUTIONS

Harmful by inhalation or if swallowed.

Please follow the manufacturer's instructions relating to the handling, use, storage and disposal of such material.

All chemicals should be considered as potentially hazardous.

We therefore recommend that this product is handled only by those persons who have been trained in laboratory techniques and that it is used in accordance with the principles of good laboratory practice.

Wear suitable protective clothing such as laboratory overalls, safety glasses and gloves. Care should be taken to avoid contact with skin or eyes. In the case of contact with skin or eyes wash immediately with water.

See material safety data sheet(s) and/or safety statement(s) for specific advice.

STORAGE

See label for storage conditions and expiry date. Please adhere to the following guidelines when handling Mini/Midi Parasep[®] and Mini/Midi Parasep[®] SF.

N.B. ParaBeads[®] must not be used with AlcorFix or Ethyl Acetate as it could cause the beads to dissolve. For prefilled AlcorFix tubes, decant fixative into liquid waste, rinse the tube, and replace with water (Mini 3ml, Midi 6ml).

PROTOCOL

The ParaBeads[®] will sink to the bottom of the vial. In order to ensure that they are fully dispersed, the vial containing ParaBeads[®] should be agitated as mentioned below:

- either by vortexing for 10 seconds;
- or by shaking vigorously 10 times before use.

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1. Once shaken, add **100µl** of the ParaBeads[®] solution to the mixing tube (containing any fixative other than AlcorFix as described above)

NB. Do not add ethyl acetate as this will dissolve the beads.

2. Screw the cap on the mixing tube and shake the fixative and ParaBeads[®] solution to mix.

3. Unscrew the cap and pipette a **20µl** sample of the solution onto a microscope slide for observation at 10x magnification. Count the number of beads in one Field of View (FOV).

4. Then take the Parasep Mixing tube containing the ParaBeads[®] and screw in the filter assembly to run the Parasep[®] procedure as per the instructions for use.

5. The ParaBeads[®] will then be concentrated at the bottom of the sedimentation cone. Pipette a

20µl sample from the pellet for mounting on a slide. Count and compare with previous FOV at 10x magnification.

For both Midi Parasep® and Midi Parasep® SF

1. Once shaken, add 200µl of the ParaBeads® solution to the mixing tube (containing any fixative other than AlcorFix as described above)

NB. Do not add ethyl acetate as this will dissolve the beads.

2. Screw the cap on the mixing tube and shake the fixative and ParaBeads® solution to mix.

3. Unscrew the cap and pipette a 20µl sample of the solution onto a microscope slide for observation at 10x magnification. Count the number of beads in one Field of View (FOV).

4. Then take the Parasep Mixing tube containing the ParaBeads® and screw in the filter assembly to run the Parasep® procedure as per the instructions for use.

5. The ParaBeads® will then be concentrated at the bottom of the sedimentation cone. Pipette a 20µl sample from the pellet for mounting on a slide. Count and compare with previous FOV at 10x magnification.

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